# beurer

## SL 70

<b>DE</b>	Schnarchstopper	
	Gebrauchsanweisung .....	2
<b>EN</b>	Snore stopper	
	Instructions for use .....	12
<b>FR</b>	Oreillette anti-ronflements	
	Mode d'emploi.....	22
<b>IT</b>	Dispositivo antirronquidos	
	Instrucciones de uso.....	32
<b>ES</b>	Dispositivo antirussamento	
	Istruzioni per l'uso.....	42
<b>TR</b>	Horlama durdurucusu	
	Kullanım kilavuzu .....	52
<b>RU</b>	Устройство против храпа	
	Инструкция по применению .....	62
<b>PL</b>	Urządzenie zapobiegające chrapaniu	
	Instrukcja obsługi.....	72

# DEUTSCH

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# ENGLISH

Read these instructions for use carefully and keep them for later use, be sure to make them accessible to other users and observe the information they contain.

**Dear customer,**

Thank you for choosing a product from our range. Our name stands for high-quality, thoroughly tested products for applications in the areas of heat, weight, blood pressure, body temperature, pulse, gentle therapy, massage, beauty, air and baby.

With kind regards

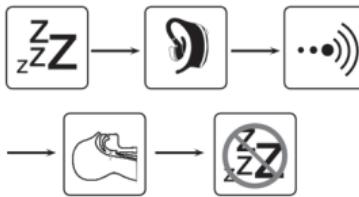
Your Beurer team

## Getting to know your device

A restorative night's sleep is of huge importance for our physical and psychological well-being; after all, it is during sleep that we process things such as new experiences and improve our mental and physical fitness. If you snore during your sleep, however, then the quality of your sleep is impaired considerably. The beurer SL 70 snore stopper was developed to reduce snoring, and to increase sleep quality as well as your well-being, for a restful night.

## How does the snore stopper work?

The snore stopper is attached to the left or right ear. The snore stopper checks for snoring noises during use. In addition, snoring produces vibrations which are transmitted via the cranial bones and which are also detected by the snore stopper. As soon as the snore stopper detects noises and vibrations, it starts sending impulses to the ear. These impulses consist of tone or tone and vibration. The intensity of these impulses increases from a low level until the desired success is achieved. The throat muscles become firm and the airways open as a reaction to the impulses. Breathing is calmed and the snoring decreases. You may also change your lying position. In the long term, there is a lasting reduction in snoring through the training of the muscles in the throat. Initially, it may take up to 7 days until you are accustomed to the device.

**The „beurer SleepQuiet“ app provides the following functions:**

- Detailed snoring analysis including snores per hour, recording duration and determination of the maximum snoring volume
- Calendar with snoring diary
- Evaluation view (in days/weeks/year)
- Adjustable impulse intensity and sensitivity

**System requirements:**

iOS ≥ 12; Android™ ≥ 8.0; Bluetooth® ab 4.0

List of supported devices:

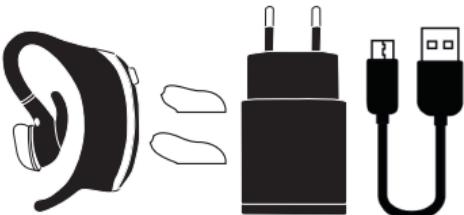
## Table of contents

1. Included in delivery .....	13	8. Cleaning and maintenance ...	17
2. Signs and symbols .....	14	9. Accessories and	
3. Intended use .....	14	replacement parts .....	18
4. Warnings and safety notes .....	14	10. What if there are problems?..	18
5. Device description.....	15	11. Disposal .....	18
6. Initial use .....	15	12. Technical specifications .....	18
7. Usage .....	17	13. Warranty / Service .....	19

## 1. Included in delivery

Check that the delivery has not been tampered with and make sure that all components are present. Before use, ensure that there is no visible damage to the device or accessories and that all packaging material has been removed. If you have any doubts, do not use the device and contact your retailer or the specified Customer Services address.

- 1x snore stopper
- 3x earplugs (small, medium, large)
- 1x micro USB charging cable
- 1x mains adapter

## 2. Signs and symbols

The following symbols are used on the device, in these instructions for use, on the packaging and on the type plate for the device:

	<b>Warning</b> Warning instruction indicating a risk of injury or damage to health
	<b>Important</b> Safety note indicating possible damage to the device/accessory
	<b>Product information</b> Note on important information
	Disposal in accordance with the Waste Electrical and Electronic Equipment EC Directive – WEEE
	Manufacturer
	<b>CE labelling</b> This product satisfies the requirements of the applicable European and national directives.
	Dispose of packaging in an environmentally friendly manner
	The products demonstrably meet the requirements of the Technical Regulations of the EAEU.

## 3. Intended use

The device may only be used for the purpose for which it is designed and in the manner specified in the instructions for use. Any form of improper use can be dangerous. The manufacturer is not liable for damage resulting from improper or incorrect use.

## 4. Warnings and safety notes

### **WARNING**

- Protect the snore stopper from impacts, humidity, dirt, marked temperature fluctuations and direct sunlight.
- Keep packaging material and the snore stopper away from children. Choking hazard!

- Keep the snore stopper away from heat sources and do not use it near (~1 m) shortwave or microwave devices.
- Use and store the snore stopper at an ambient temperature between 0°C and 60°C. Higher or lower temperatures may damage the snore stopper.
- The snore stopper is only splash-proof.
- Keep the snore stopper away from water.
- Do not use any chemical or abrasive cleaning agents. Clean the snore stopper occasionally with a 70% isopropyl alcohol pad.
- If your skin or eyes come into contact with fluid from a rechargeable battery cell, flush out the affected areas with water and seek medical assistance.
- Protect batteries from excessive heat.
- Risk of explosion! Never throw batteries into a fire.
- Do not disassemble, split or crush the rechargeable batteries.
- Fully charge the battery prior to initial use.
- Fully charge the battery at least every 3 months.
- Batteries must be charged correctly prior to use. The instructions from the manufacturer and the specifications in these instructions for use regarding correct charging must be observed at all times.

## 5. Device description

1 ON/OFF button

2 Operation light

Green = battery at 41–100%  
Red = battery at 0–40%

3 Bluetooth® light

4 Micro USB connection

5 Earplugs

## 6. Initial use

### Connecting the snore stopper to the “beurer SleepQuiet” app

You can also use the basic functions of the snore stopper without the “beurer SleepQuiet” app. The additional features such as recording, statistics, calendar and settings are only available in conjunction with the “beurer SleepQuiet” app.

To connect your snore stopper with the “beurer SleepQuiet” app, please proceed as follows:

1. Ensure that your smartphone is connected to the internet.
2. Activate *Bluetooth®* on your smartphone.
3. Download the free “beurer SleepQuiet” app from the Apple App Store (iOS) or at Google Play (Android).

Depending on the settings for your smartphone, additional charges may be incurred for connecting to the Internet or transferring data.

4. Start the “beurer SleepQuiet” app and follow the instructions.

## Attaching the snore stopper

You can attach the snore stopper to the left or right ear.

Here, it is important to turn the earplug [5] so that the opening is facing the direction of the ear canal once attached.

- Attach the snore stopper to your ear as shown. Do **not** bend the upper clip on the snore stopper too strongly as it may become damaged.

1.

2.

3.

If the earplug easily moves in your ear, try one of the other earplugs included in delivery for optimum fit.

For individual adjustment there is the option of carefully shortening the small hook on the device using a pair of scissors. When doing so, it is important to only cut off a small amount at a time and to repeat this, if necessary, until you have made the hook the right size. Check whether it is a good fit after each cut. To make it comfortable to wear, please ensure that the cuts are even and round off the corners.

Please ensure that the hook is securely connected to the device and note that it cannot be replaced.

## **7. Usage**

1. Press and hold the ON/OFF button on the snore stopper for three seconds. The operation light begins to light up. The snore stopper is now switched on. If the snore stopper cannot be switched on, the battery in it is flat. Recharge the battery as described in Chapter "7.1 Charging the battery". After a short time, the operation light switches off so that it does not disturb you in the night.
  2. Attach the snore stopper to your left or right ear as shown.
  3. Go to bed.
  4. As soon as you get up, switch off the snore stopper. To do this, press and hold the ON/OFF button on the snore stopper for three seconds. A short vibration confirms that the snore stopper has been successfully switched off.
- Initially, it may take up to 7 days until you are accustomed to the device.

### **Charging the battery**

1. Open the cover on the micro USB connection of the snore stopper.
2. Connect the charging cable included in delivery to the snore stopper and a computer that is switched on. Alternatively, connect the charging cable with the snore stopper and the mains adapter. Insert the mains adapter into a suitable socket.
3. Whilst charging, the operation light flashes green. As soon as the battery is fully charged, the operation light will remain green.
4. It usually takes approx. 3 hours to fully charge the battery. As soon as the battery is completely charged, remove the charging cable from the computer or the socket and the mains adapter and the snore stopper.
5. When the battery is fully charged, the battery life is approx. 4–5 days. The operation light turns red as soon as the battery capacity is below 41%.

### **Transferring data to your smartphone**

To transfer the recorded data from your snore stopper to the "beurer SleepQuiet" app, proceed as follows:

1. Activate *Bluetooth®* on your smartphone.
2. Open the "beurer SleepQuiet" app. Press the ON/OFF button on the snore stopper. The recorded data is now transferred to the smartphone. For a large volume of data, this may take some time.

## **8. Cleaning and maintenance**

Do not use any chemical or abrasive cleaning agents. Clean the snore stopper occasionally with a 70% isopropyl alcohol pad.

## **9. Accessories and replacement parts**

To purchase accessories and replacement parts, please visit [www.beurer.com](http://www.beurer.com) or contact the corresponding service address (according to the service address list) in your country. Accessories and replacement parts are also available from retailers.

Designation	Item number and/or order number
Set of earplugs (3 sizes, silicone)	163.910

## **10. What if there are problems?**

Problem	Possible cause	Solution
Device does not switch on	Battery dead	Charge the battery
Device does not switch on and battery is fully charged	Device defective	Contact Customer Services

## **11. Disposal**

For environmental reasons, do not dispose of the device in the household waste at the end of its useful life.

Dispose of the device at a suitable local collection or recycling point.

Dispose of the device in accordance with EC Directive – WEEE (Waste Electrical and Electronic Equipment).

If you have any questions, please contact the local authorities responsible for waste disposal.

## **12. Technical specifications**

Dimensions	49 x 15 x 62 mm
Weight	15 g (without earplugs)
Connection	Micro USB connection
Mains adapter voltage supply	Input: 100-240 V~; 50/60 Hz; 0.3 A Output: 5 V == 1000 mA

Mains adapter protection class	<input checked="" type="checkbox"/> double-protected
Battery	80 mAh lithium polymer
Battery life	Approx. 4 days (depending on use)
Memory	1 night for heavy snoring
Material	Silicone
Data transfer	Bluetooth® low energy technology
Permissible operating conditions	0°C to 60°C, 30 to 75 % RH 860 to 1060 hPa ambient pressure
Permissible transport and storage conditions	-20°C to 60°C, <75 % RH 860 to 1060 hPa ambient pressure
Frequency band	2402 – 2480 MHz
Transmission power	3.7 dBm

Subject to technical changes.

We hereby confirm that this product complies with the European RED Directive 2014/53/EU.

The CE Declaration of Conformity for this product can be found under:  
<https://www.beurer.com/web/we-landingpages/de/cedeclarationofconformity.php>

## 13. Warranty / Service

Beurer GmbH, Söflinger Straße 218, 89077 Ulm, Germany (hereinafter referred to as "Beurer") provides a warranty for this product, subject to the requirements below and to the extent described as follows.

**The warranty conditions below shall not affect the seller's statutory warranty obligations which ensue from the sales agreement with the buyer. The warranty shall apply without prejudice to any mandatory statutory provisions on liability.**

Beurer guarantees the perfect functionality and completeness of this product.

The worldwide warranty period is 3 years, commencing from the purchase of the new, unused product from the seller.

The warranty only applies to products purchased by the buyer as a consumer and used exclusively for personal purposes in the context of domestic use. German law shall apply.

During the warranty period, should this product prove to be incomplete or defective in functionality in accordance with the following provisions, Beurer shall carry out a repair or a replacement delivery free of charge, in accordance with these warranty conditions.

**If the buyer wishes to make a warranty claim, they should approach their local retailer in the first instance: see the attached “International Service” list of service addresses.**

The buyer will then receive further information about the processing of the warranty claim, e.g. where they can send the product and what documentation is required.

A warranty claim shall only be considered if the buyer can provide Beurer, or an authorised Beurer partner, with

- a copy of the invoice/purchase receipt, and
- the original product.

The following are explicitly excluded from this warranty:

- deterioration due to normal use or consumption of the product;
- accessories supplied with this product which are worn out or used up through proper use (e.g. batteries, rechargeable batteries, cuffs, seals, electrodes, light sources, attachments and nebuliser accessories);
- products that are used, cleaned, stored or maintained improperly and/or contrary to the provisions of the instructions for use, as well as products that have been opened, repaired or modified by the buyer or by a service centre not authorised by Beurer;
- damage that arises during transport between manufacturer and customer, or between service centre and customer;

- products purchased as seconds or as used goods;
- consequential damage arising from a fault in this product (however, in this case, claims may exist arising from product liability or other compulsory statutory liability provisions).

Repairs or an exchange in full do not extend the warranty period under any circumstances.

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The *Bluetooth*® word mark and logos are registered trademarks owned by Bluetooth SIG, Inc. and any use of such marks by Beurer GmbH is under license. Other trademarks and trade names are those of their respective owners.

Apple and the Apple logo are trademarks of Apple Inc., registered in the U.S. and other countries. App Store is a service mark of Apple Inc., registered in the U.S. and other countries.

Google Play and the Google Play logo are trademarks of Google LLC. Android is a trademark of Google LLC.

Beurer GmbH • Söflinger Straße 218 • 89077 Ulm, Germany  
[www.beurer.com](http://www.beurer.com) • [www.beurer-gesundheitsratgeber.com](http://www.beurer-gesundheitsratgeber.com)  
[www.beurer-healthguide.com](http://www.beurer-healthguide.com)