


- D** Kamera einstecken
- G** Plug in camera
- F** Brancher la caméra
- E** Insertar cámara
- I** Inserire la videocamera



- D** Schalter auf „ON“ schieben
- G** Set switch to „ON“
- F** Glisser le bouton sur « ON »
- E** Colocar el interruptor en posición „ON“
- I** Spostare l'interruttore su „ON“

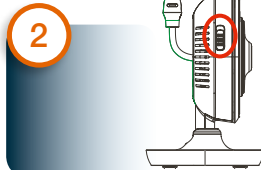


- D** „Beurer CareCam“ im App Store/Google Play Store installieren
- G** Install „Beurer CareCam“ from the App Store/Google Play Store
- F** Installer «Beurer CareCam» depuis l'App Store/Google Play Store
- E** Instalar „Beurer CareCam“ de App Store/Google Play Store
- I** Installare „Beurer CareCam“ scaricandola dall'App Store/da Google Play Store



- D** App starten und den Anweisungen folgen
- G** Start the app and follow the instructions
- F** Démarrer l'application et suivre les instructions
- E** Iniciar la aplicación y seguir las instrucciones
- I** Avviare la app e seguire le istruzioni



- D** BY 88/99 in App auswählen
- G** Select BY 88/99 in the app
- F** Sélectionner BY 88/99 dans l'application
- E** Seleccionar BY 88/99 en la aplicación
- I** Nella app selezionare BY 88/99

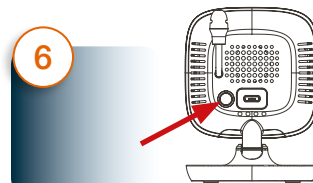

- D** Nach Aufforderung der App die „PAIR“-Taste 3 Sek. drücken
- G** Press the „PAIR“ button for 3 sec when requested to do so by the app
- F** Appuyer sur la touche « PAIR » à la demande de l'application
- E** Cuando la aplicación lo solicite, pulsar la tecla „PAIR“ durante 3 segundos
- I** Quando richiesto, premere il pulsante „PAIR“ per 3 s



- D** Kamera über WLAN des Endgeräts verbinden. Anweisungen folgen
- G** Connect camera via the WLAN of the end device. Follow the instructions
- F** Connecter la caméra à l'aide du WiFi de l'appareil. Suivre les consignes
- E** Conectar la cámara a través de la WLAN del equipo terminal. Seguir las instrucciones
- I** Collegare la videocamera mediante la WLAN del dispositivo. Seguire le istruzioni




- D** Livebild erscheint
- G** A live image appears
- F** L'image en direct s'affiche
- E** Aparece la imagen en vivo
- I** Compare l'immagine live

- D** Funktionen während der Liveübertragung: Tippen Sie dazu auf das Übertragungsbild
- GB** Functions during live transmission: tap the transmission image
- F** Fonctions pendant la transmission en direct de l'image: Appuyez sur l'image actuelle
- E** Funciones durante la transmisión en directo: Pulse el terminal la imagen que está transmitiendo la cámara
- I** Funzioni durante la trasmissione live: Premere sull'immagine trasmessa




- D** Gegensprechen
- GB** Intercom
- F** Interphone
- E** Intercomunicación
- I** Comunicazione bidirezionale

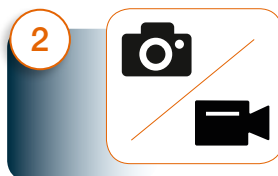

- D** Fotos/Filme (Android)
- GB** Photos/films (Android)
- F** Photo/film (Android)
- E** Hacer foto/filmar (Android)
- I** Foto/filmati (Android)



- D** Schlaflieder
- GB** Lullabies
- F** Berceuses
- E** Nanas
- I** Ninnenanne

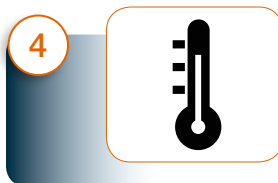

- D** Temperaturanzeige (°C/°F)
- GB** Temperature display (°C/°F)
- F** Affichage de la température (°C/°F)
- E** Indicación de temperatura (°C/°F)
- I** Indicazione della temperatura (°C/°F)



Service Hotline
+49 (0) 731 / 39 89 - 144

Beurer GmbH
Söflinger Straße 218
89077 Ulm, Germany
www.beurer.com
email: kd@beurer.de